JABATAN PATOLOGI, HSAJB

**POLISI DAN PROSEDUR
PENGENDALIAN
SAMPEL COVID-19
Versi 01 (12 APRIL 2020)**

**DISAHKAN OLEH:
DR NORIDA KHALID
KETUA JABATAN PATOLOGI HSAJB**

BIL	KANDUNGAN	MUKA SURAT
1.	Pengenalan Umum	3
2.	Tatacara Pengendalian Sampel COVID 19 / Severe Acute Respiratory Infection (SARI) / Patient Under Investigation (PUI) ke Makmal Mikrobiologi Hospital Sultanah Aminah Johor Bahru	4-10
3.	Tatacara Pengendalian Sampel COVID 19 / Severe Acute Respiratory Infection (SARI) / Patient Under Investigation (PUI) ke Makmal Hematologi Hospital Sultanah Aminah Johor Bahru.	11-13
4.	Tatacara Pengendalian Sampel COVID 19 / Severe Acute Respiratory Infection (SARI) / Patient Under Investigation (PUI) ke Patologi Kimia Hospital Sultanah Aminah Johor Bahru.	14-16
5.	Tatacara Pengendalian Sampel COVID 19 / Severe Acute Respiratory Infection (SARI) / Patient Under Investigation (PUI) ke Makmal Patologi Anatomi Hospital Sultanah Aminah Johor Bahru.	17-19
6.	Lampiran 1	20
7.	Lampiran 2	21

PENGENALAN UMUM

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MOLECULAR TEST

1. Covid-19 PCR
2. Influenza PCR (if required)

NON-MOLECULAR TEST

1. Culture
2. Serology (Rapid test: HIV; Dengue NS1/IgM/IgG)
3. Urine FEME : Dipstick only

*Serum COVID-19: will be sent to IMR by batch via postage

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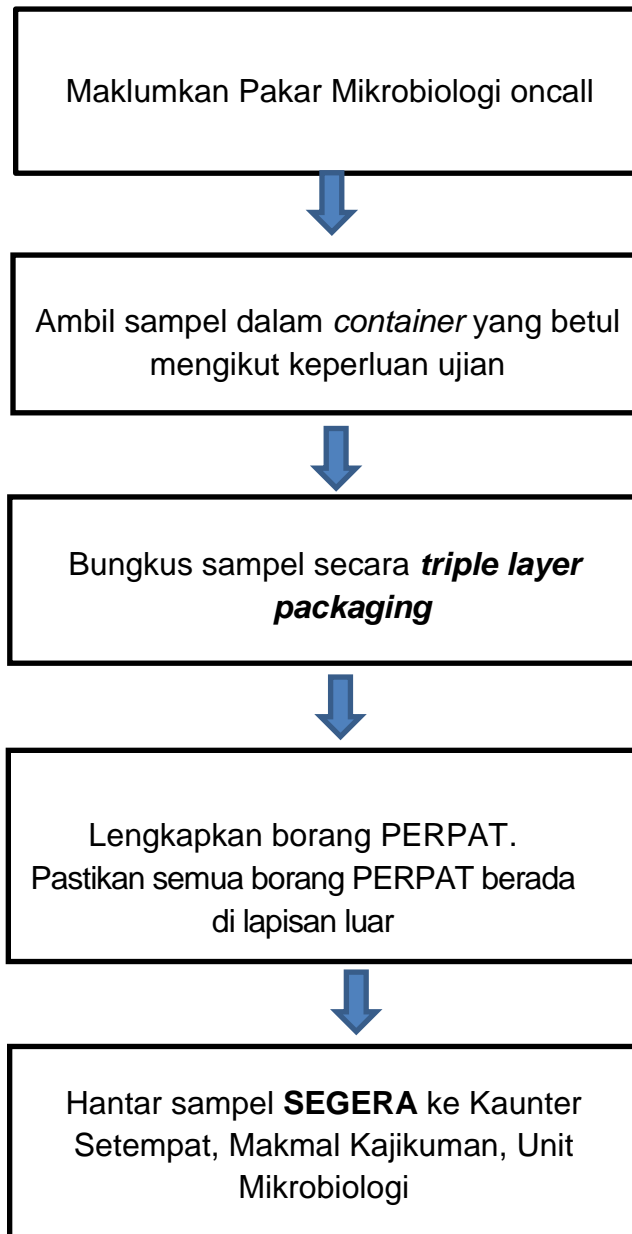
SPECIMEN CONTAINER COVID-19 PACKAGING FOR HEMATOLOGY UNIT

TEST REQUEST	SPECIMEN TYPE
Full Blood Count - Adult (3ml) (EDTA Tube - Purple Cap)	
Full Blood Count - Pediatric (500uL) (EDTA Tube - Purple Cap)	
PT/APTT/ INR/Fibrinogen (Sodium Citrate Tube - Blue Cap)	
G6PD (Dried Blood Spot On Filter Paper)	

Carta Alir : Penghantaran sampel darah COVID 19 / Severe Acute Respiratory Infection (SARI) / Patient Under Investigation (PUI) ke Makmal Hematologi Hospital Sultanah Aminah Johor Bahru.

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SPECIMEN CONTAINER COVID-19 PACKAGING FOR CHEMICAL PATHOLOGY UNIT

TEST REQUEST	SPECIMEN TUBE
<p>Routine Biochemistry</p> <ol style="list-style-type: none"> 1. Renal Profile (RP) 2. Liver Function Test (LFT) 3. C-reactive protein (CRP) 4. Ferritin 	<p>Green Cap / Lithium Heparin Tube</p> 
<p>Therapeutic Drug Monitoring (TDM)</p>	<p>Red Cap / Plain tube</p> 
<p>Therapeutic Drug Monitoring (TDM)</p> <p>ONLY FOR:</p> <ol style="list-style-type: none"> 1. Cyclosporine 2. Tacrolimus 	<p>Purple Cap / EDTA tube</p> 

Arterial/ venous blood gas

- Analysis of ABG/VBG for SARI/ PUI/ Covid -19 patients are done using the analyser located in PICU (BR2).
- For analysis of ABG/ VBG please make sure the following factors are done correctly.
 1. The syringe must be coated with adequate volume of anticoagulant (heparin).
 2. After withdrawal of blood, the syringe must be inverted, bubbles of air discarded and the syringe must be capped using a stopper (Do send the syringe with the needle).
 3. ABG/VBG must be transported in slurry ice and analysed immediately.
- Please follow the 'Garis panduan pengambilan ABG/VBG' which can be viewed from HSAJB website).

**Tatacara Pengendalian Sampel
COVID 19 / Severe Acute
Respiratory Infection (SARI) /
Patient Under Investigation
(PUI) Ke Makmal Patologi
Anatomi.**

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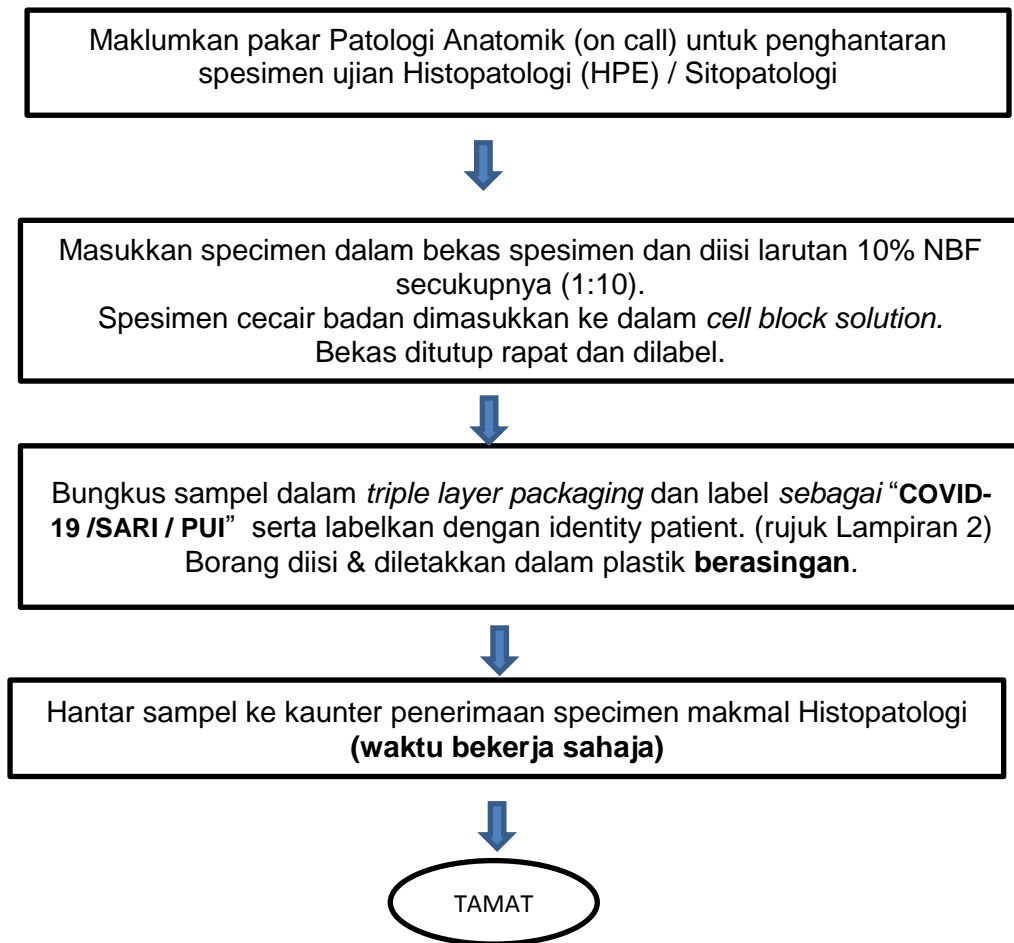
PACKAGING LAYER	
SPECIMEN TUBE	
<ul style="list-style-type: none">• Use correct type of container for the test requested (Ex: Tube/ Sterile Container / Blood Culture Bottle• Label with patients name and IC• Make sure the volume of blood taken is sufficient.• Decontaminate by wiping the surface with 70% alcohol.• The tube does not need to be sealed	
PRIMARY BIOHAZARD BAG <ul style="list-style-type: none">- Place tube into leak-proof sealed biohazard bag.- Decontaminate by wiping the surface with 70% alcohol.	
SECONDARY BIOHAZARD BAG <ul style="list-style-type: none">- Place primary container into leak-proof sealed biohazard bag.- Decontaminate by wiping the surface with 70% alcohol.	
TERTIARY BIOHAZARD BAG <ul style="list-style-type: none">- Place secondary container into leak-proof sealed biohazard bag.- Decontaminate by wiping the surface with 70% alcohol	
POLISTERENE BOX <ul style="list-style-type: none">- Place the tertiary container into the polisterine box.- The box does not need to be sealed.	
<ul style="list-style-type: none">- Place Perpat - 301 form into a separate biohazard bag with the details facing on the clear side for easy viewing of types of test.- make sure correct tube is paired with the correct form and sent to the correct laboratory.- Place biohazard bag with the PER PAT 301 forms inside it on top of the polisterine box.	

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Packaging	Cytology	Histopathology
Primary container	Cell block container <ul style="list-style-type: none">- Labelled with patient Name and IC- Leak proof seal- Decontaminate with 70% alcohol	Specimen container <ul style="list-style-type: none">- Labelled with patient Name and IC- Leak proof seal- Decontaminate with 70% alcohol
Secondary container	Leak-proof sealed bag <ul style="list-style-type: none">- Place in biohazard plastic bag	Leak-proof sealed bag <ul style="list-style-type: none">- Place in biohazard plastic bag
Tertiary container	Leak-proof sealed bag <ul style="list-style-type: none">- Place in biohazard plastic bag- Decontaminate with 70% alcohol- Labelled with patient Name and IC- Label “COVID-19 /SARI / PUI	Leak-proof sealed bag <ul style="list-style-type: none">- Place in biohazard plastic bag- Decontaminate with 70% alcohol- Labelled with patient Name and IC- Label with COVID-19/ /SARI / PUI
PER-PAT 301	<ul style="list-style-type: none">- Label as “COVID-19 /SARI / PUI. Put form in separate biohazard bag	<ul style="list-style-type: none">- Label as “COVID-19 /SARI / PUI. Put form in separate biohazard bag